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EN Symbols on the packaging: These symbols indicate:

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The atmospheric pressure limitation for operation, transport and storage.

Luftdruckbeschränkung für Betrieb, Transport und Lagerung. La limite de pression atmosphérique à ne pas dépasser en cours de fonctionnement, ainsi que pour le transport et le stockage. I limiti di pressione atmosferica per funzionamento, trasporto e conservazione. Luchtdrukbeperkingen bij bediening, transport en opslag. Gränsvärden för atmosfäriskt tryck under drift, transport och förvaring av apparaten. La limitación de la presión atmosférica para el funcionamiento, transporte y conservación.

To handle the fragile device with care. Vorsicht, zerbrechlich. L'appareil est fragile et doit être manipulé avec soin. Il dispositivo è fragile e deve essere maneggiato con attenzione. Het apparaat met zorg behandeld moet worden. Apparaten är ömtålig och ska hanteras försiktigt. Maneje este dispositivo frágil con cuidado.

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
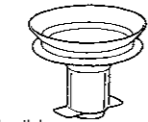
1

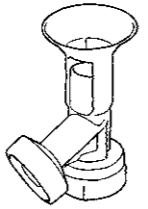Frame
Houder
Tratthållaren
Marco


Instructions for use Comfort-Breastshield

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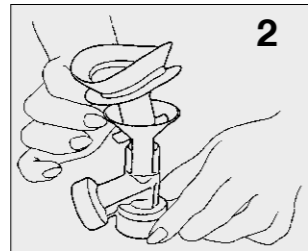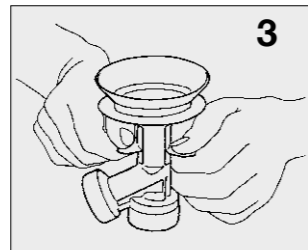
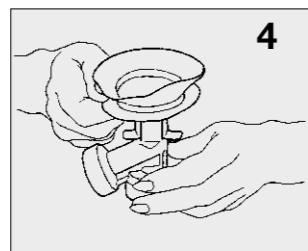
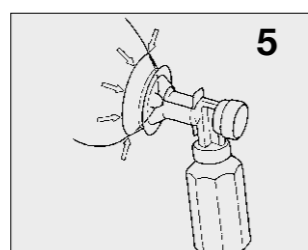
5

EN Comfort Breastshield

Medela's Comfort Breastshield is particularly kind to the body and, in conjunction with Medela's breastpumps, it is suitable for the convenient and pleasant pumping of breastmilk. Together with the natural physiological suction rhythm of a Medela breastpump, it simulates breastfeeding in an ideal manner.

1. The Comfort Breastshield consists of two parts: the frame and the flexible shield (see drawing no. 1). Please make sure that both parts are dry, so that the flexible shield will be well held in the frame.
2. Place the frame on the table with the funnel-shaped part pointing upwards. Insert the tags on the tubular part of the shield into the apertures on the side of the funnel tube (see drawing no. 2). Make sure that the inner part of the shield tube slides into the funnel tube.
3. Pull the tags down on both sides until they fit to the base (see drawing no. 3).
4. Pull down the outer ring of the shield over the edge of the funnel (see drawing no. 4). Pull the tags down again (see drawing no. 3) to ensure that the inner part of the shield is in close contact.

Important

First place the Comfort Breastshield in position on the breast (with the nipple in the centre). It is especially important to make sure the shield seals to the breast (see drawing no. 5).

Afterwards switch on the breastpump. Hold the breastshield on the tubular part in place using your index and middle finger.

Separate and Cleaning

Frame and flexible shield can easily be separated under running water (the water works like grease). Wash in a dishwasher or boil for approximately 3 minutes (add a little household vinegar to the water to prevent limescale deposits).

Caution:

The use of microwave to heat up mother's milk is not recommended. Uneven heating may cause the bottle to feel warm, but certain portions of the milk may become extremely hot. In some cases, hot enough to cause injury to the baby's mouth.

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